## STANDING TRANSFER STAND & TURN FRAME - FULL FRAME

- Carer to ensure correct positioning of stand and turn frame, and chair.
   Carer to ensure locking pin is secured in position prior to use.
- Person to shuffle bottom to front of seat.
  - Person to position feet on base plate.
  - Carer to hold onto frame.
  - Person to hold frame, lean forward with head raised and stand.
- Once person is standing, carer to release locking pin and swivel frame.

- Carer to re-engage locking pin.
  - Person to sit down.
  - Once seated, carer to remove frame.



90°

