
A9

STANDING TRANSFER STAND & TURN FRAME – WHEELED WITH SEAT

- 1
- Person to shuffle bottom to front of seat.
- Move frame in front of person.
- Person to position feet on base plate.
- · Carer to apply brakes.
- Person to hold frame, lean forward with head raised and stand.



- 2
- Once person is standing, carer to lower both seat paddles.
- Person to sit.
- Carer to release brakes and manoeuvre frame.

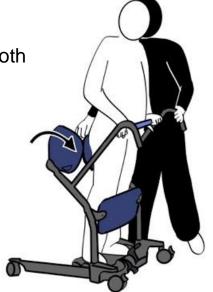

- 3
- Person to be positioned so that back of knees touch seat / bed.
- Carer to apply brakes.
- Person to stand.
- Carer raises both seat paddles whilst person maintains standing.
- Person to sit whilst holding onto frame.
- Once seated, carer to remove frame.



N.B. • Legs on frame can be opened to enable positioning around chair legs.



