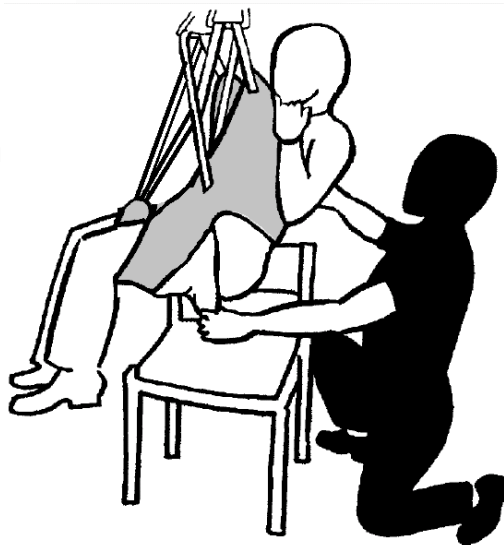


**1**


- Undo fastening of trousers/belt/skirt etc.
- Position clothing down as far as possible.
- Fit sling as per Moving and Handling Care Plan.

**2**

- When hoisted, position clothing down further over the buttocks.

**3**


- Hoist down onto commode/toilet.
- Remove leg sections of sling.
- Lower clothes further down over the knees.
- Remove continence pad (if worn).

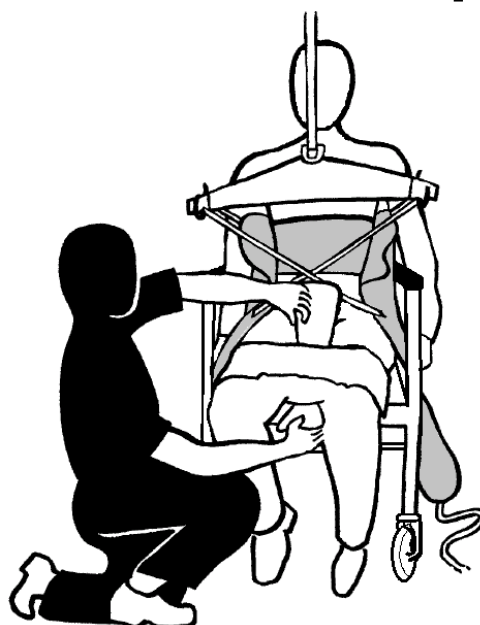

**4**

After toileting

- Position continence pad (if worn).
- Position clothing up legs as high as possible.
- Re-fit leg sections of sling below gathered clothes.

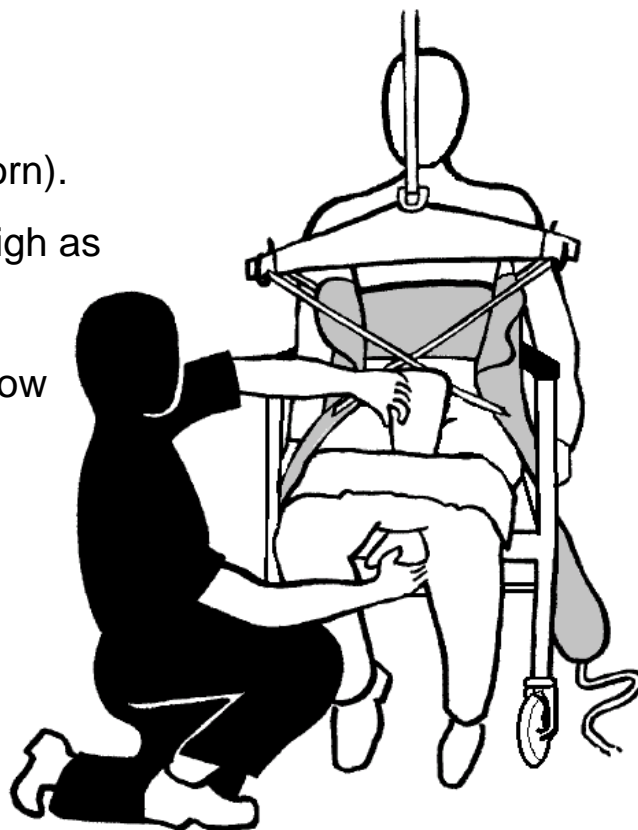**5**

- Hoist person.
- Adjust continence pad into position.
- Position clothes up over buttocks.
- Hoist person back down, remove sling and fasten clothing.

