


Extension straps will have varying number of loops depending on manufacturer.

Always refer to Moving and Handling Care Plan (MHCP) for use of extension straps


Sling extension straps can be used on a variety of slings.

They should never be applied to X straps.

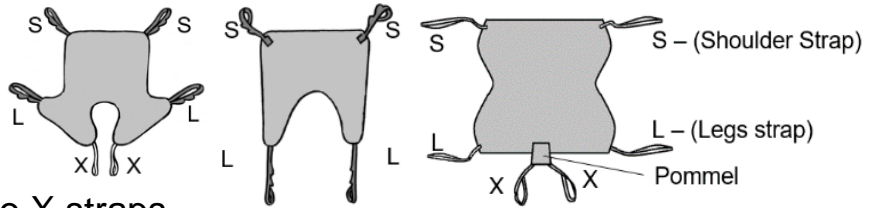


FITTING INSTRUCTIONS

- 1** Extension strap to be threaded (top section first) through the top loop of sling strap.

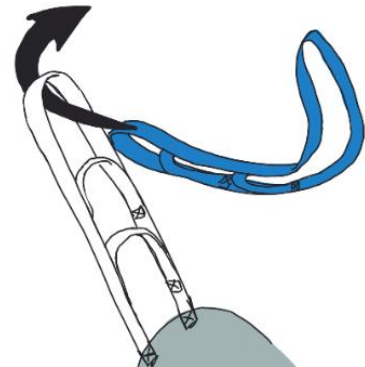

- 2** Top of extension strap to be thread through bottom section of extension strap.

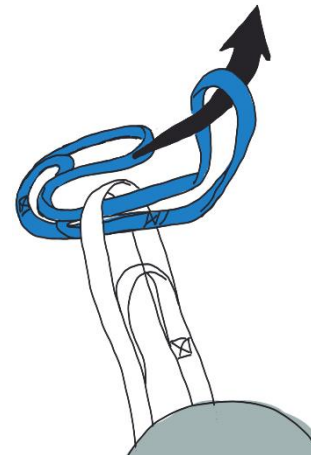

- 3** Pull extension strap through.
Ensure that connection is tight.
Repeat for all straps as required.
Ensure straps are secure before hoisting.
Refer to MHCP for correct loop settings.

