


# FITTING SLIDE SHEET

## WITHOUT ROLLING THE PERSON

### 2 CARERS

1

Carer to fold and roll slide sheet.  
Until almost entirely folded.  
(If using a Looped Slide Sheet,  
start fold from the closed edge.)

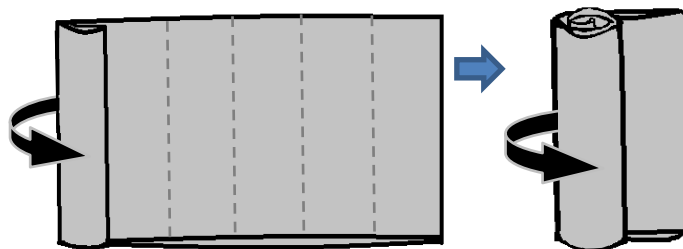

2

Carer to place Slide Sheet (with rolled section downwards against the bed) under persons head, sliding underneath pillow, or under persons neck.



3

Carers to stand either side of the bed.  
Carers to unroll folds of slide sheet, one section at a time. Carers to co-ordinate movements, with verbal commands such as “ready, slide”.



4

Repeat, keeping movements small, until slide sheet is completely unrolled.



- N.B.**
- If bed side rails are in situ, lower prior to fitting slide sheet.
  - Ensure bed is at appropriate height for carers.